K-Active® ELEKTROLYT CREME

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Inhaltsstoffe (INCI): Trace Element Solution (Aqua purificata, Zinc, Iron), Octyldodecanol, Hydrogenated Palm Glycerides, PEG-200 Hydrogenated Glycerol Palmate, Cetyl Alcohol, Methylparaben (and) Sorbic Acid (and) Dehydroacetic Acid (and) Propylparaben

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K-Active® ELEKTROLYT CREAM

Medical Device

K-Active® ELEKTROLYT CREAM is a wound healing cream for injuries with the following characteristics:

- Promotes wound healing
- Reduces swelling
- Relieves pain
- Reduces scar formation
- Prevents infections
- Doping free.

Areas of application:

Injuries of the skin (e.g. cuts, abrasions, lacerations and contusions), also at the mouth and on mucous membranes, burns 1st and 2nd degree (also sunburns), blunt traumas (e.g. bruises, strains) and scars. K-Active® ELEKTROLYT CREAM is also suitable for children and sensitive, stressed or dry, cracked and aging skin.

Characteristics:

Bruising and edema are accelerated degraded, reduces the swelling and thus relieves pain. Formation of new tissue is supported, improves healing after surgery or injury (wounds, burns, sunburn) and reduces the scarring.

Application:

Before and after the treatment of a wound wash the hands and, if necessary, for example, clean with K-Active® FIRST AID SPRAY carefully. Apply K-Active® ELEKTROLYT CREAM approximately 2 mm thick on the injured skin to approximately 2 cm above the wound edges out. In the first three days after the injury apply several times a day, afterwards use as needed.

K-Active® ELEKTROLYT CREAM can also be applied in combination with a conventional bandaging material. By covering the injured skin with a bandage the wound is additionally protected from external influences that may interfere with the healing. K-Active® ELEKTROLYT CREAM is suitable for iontophoresis. It is applied under the anode.

Period of application:

The application depends on type and severity of the injury.

Contraindications and side effects:

Contraindications and side effects are not known. With a known allergy or suspected allergic reactions to any of the ingredients, the product should not be used.

Precautions:

For external use only. Protect from frost and heat. Ensure hygienic removal. Close tube after use. Ensure undamaged seal. Expiry date: see the bottom and tube fold. Do not use after the expiration date.

Ingredients (INCI): Trace Element Solution (Aqua purificata, Zinc, Iron), Octyldodecanol, Hydrogenated Palm Glycerides, PEG-200 Hydrogenated Glycerol Palmate, Cetyl Alcohol, Methylparaben (and) Sorbic Acid (and) Dehydroacetic Acid (and) Propylparaben

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